AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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en english

Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

VenoTrain ulcertec is a medical device used for the long-term treatment of severe phlebological disorders in the legs.

Indications

AD indications:

- Active venous leg ulcer without edema (CEAP stage 6)
- Prevention of recurring venous leg ulcers (CEAP stage 5)

AG indications:

- Active venous leg ulcer with a tendency to edema formation
- Venous leg ulcer with edema in the thigh
- Venous leg ulcer with varicose veins in the thigh
- Post-thrombotic syndrome on more than one level (pelvis and / or thigh and / or lower leg)

Part of the body this product is used for

On the leg, in accordance with the indications

Risks of using this product

⚠ Caution¹

There are no known application risks if the product is used properly, provided your legs have been measured by a trained specialist and you have been taught the correct method for putting on the compression stockings.

Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.

- Avoid contact with ointments, lotions, or substances containing grease or acids.
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor.

Contraindications

- Advanced peripheral artery disease (if one of these parameters is met: ABPI < 0.5, arterial pressure at the ankle < 60 mmHg, toe pressure < 30 mmHg, or tcpO₂ < 20 mmHg at the instep).
- Decompensated heart failure (NYHA III + IV)
- Septic phlebitis
- Phlegmasia coerulea dolens

In the following cases, the decision to treat should be made after weighing the benefits

against the risks in order to choose the most appropriate means of compression:

- Dermatitis with significant weeping
- Intolerance of the compression material
- Severely impaired sensitivity of the extremity
- Advanced peripheral neuropathy (e. g. in the case of diabetes mellitus)
- Rheumatoid arthritis

Application instructions

- The initial fitting must only be performed by medically trained specialists.
- It is best to put on your stockings first thing in the morning, as soon as you get up.
- Do not use the compression stockings without a wound dressing.
- Wear Bauerfeind gloves to put on and take off your VenoTrain compression stockings as this makes it easier to distribute the material evenly over the leg.
- Your legs and hands must be dry.
- Do not grasp the knitted fabric with sharp fingernails. Make sure that you take off your jewelry before starting.
- Avoid damaging the knitted fabric and do not cut off any loose threads or labels, as this could cause stitches made during production to unravel.
- Store your VenoTrain compression stockings in a dry place away from direct light.
- The product comes as a set comprising two white liners exerting a low level of compression and a skin-colored overstocking offering a high level of compression. Since the white liner also holds the wound dressing in place, it is advisable to wear it at night too.
- Compression stockings are articles for daily use, and as such they are subject to natural wear when used on a regular basis. The recommended length of use is six months. This is based on correct handling (putting on and removal, care, etc.). The product's maximum useful life under proper storage conditions is indicated next to an "hourglass" symbol printed on the label on the packet. Custom-made medical compression stockings are intended for immediate use and are therefore not given such a date.
- The edges of the overstocking and the liner must not be positioned directly on top of each other.

Cleaning instructions

- Please use mild detergent and a lingerie bag. Never expose your product to direct heat. Also observe the instructions on the sewn-in label on the upper edge of your product. Regular care guarantees optimum effectiveness of the product.
- The product can be machine dried provided a delicate cycle is chosen. **Also observe the instructions on the sewn-in label on the upper edge of your product.**
- When handwashing the stockings, we recommend drying them between two towels (do not wring).

- Avoid the use of washing additives like fabric softener, optical brighteners or stain removers.
- Avoid cleaning with chemicals and ironing.

Notes on reuse

This product is intended for your personal use only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the VenoTrain ulcer tec have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical product to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Version: 2020-02

¹ Indicates the risk of personal injury (risk of injury, harm to health, accidents and potentially also damage to property)

Tips for putting on the compression stockings

- A ▶ After applying the wound dressing, gather the white **liner** over your toes.
- B ▶ Stretch the opening for the foot with both hands. Now slip the stocking on until it covers the heel.
- C ▶ When you pull it upward, the stocking should be placed carefully over the wound dressing.
- D1 ▶ Then pull the stocking evenly upward to below the knee.
- D2 ▶
- E ▶ To make it easier to put on the **overstocking**, we recommend gathering it up and stretching it horizontally.
- F ▶ Then slip the opening of the stocking over your toes as usual.

- G ▶ Once you have put the stocking on your foot, check whether the heel fits correctly.
- H ▶ You should now pull the stocking upward as far as possible until it reaches the hollow of the knee.
- I1 ▶ This will ensure that it fits correctly and has an optimum medical effect.
- I2 ▶ Make sure that the edging of the overstocking is positioned beneath the band of the liner.
When you put on the overstocking, we recommend that you use the blue donning aid provided.

Please refer to the information provided on page 2 of this brochure.

For more details, please consult your physician or medical retailer.

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